JAY Shape Back™ Back

Important Consumer Information

NOTE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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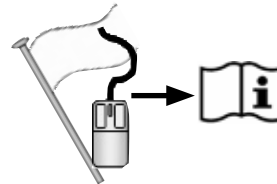
12

GENERAL WARNINGS/INTENDE USE

Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a JAY Shape Back is appropriate.

WARNING!

- Backs should only be installed by:
- An authorized Sunrise Medical supplier.
- A qualified and competent healthcare professional who has undergone appropriate certified training by either sunrise medical or their approved supplier.

THE JAY JAY SHAPE BACK INTENDED USE

The JAY SHAPE Back is designed to provide the important benefits of a proper back support and additional comfort for clients with spinal deformities such as kyphosis (curvature of the spine). It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is light-weight, durable and easy to use.

Maximum user weight: 135 kg (297.5 lbs).

The intended lifetime of the back rest is 2 years.

WARNING!

- Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.
- Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

WARNING!

Do not use JAY Shape Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

WARNING!

Do not use the JAY SHAPE back cover or straps to lift wheelchair.

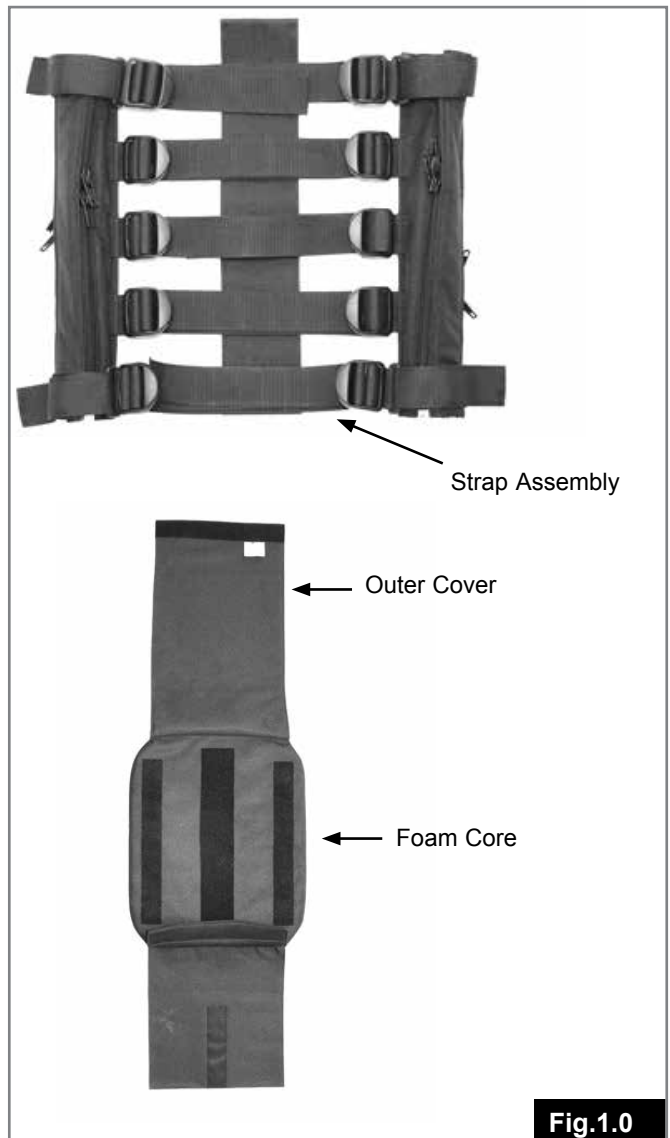
WARNING!

Carelessness causes fire.

WARNING!

Please read this manual before use.

JAY SHAPE BACK COMPONENTS

INSTALLATION

Wheelchair Types:

The JAY Shape Back is designed to be compatible with most wheelchairs with the following exceptions: Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.

WARNING!

Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.

Installation:


To install the JAY Shape Back, first remove the outer cover from the strap assembly. Then install the strap assembly onto the back canes of the wheelchair by opening the zippers around the back canes and closing the zippers again fully, (Fig.2.0).

CAUTION!

Note the height of the back. Make sure the strap assembly is appropriate to the top of the back canes and leaves enough room underneath the strap assembly for the cushion.

NOTE: Align grommet holes of the JAY Shape back with the grommet holes (if present) on the wheelchair. If provided, install the wheelchair back cane screws as shown in, (Fig.3.0).

- Ensure the buckles of the adjustment straps are facing rearward.
- Adjust each strap individually to achieve the required tension.
- Then install the cover making sure the padded side of the cover is next to the back of the client. Lastly, fold the back flap of the cover over the back of the strap assembly, (Fig.4.0).

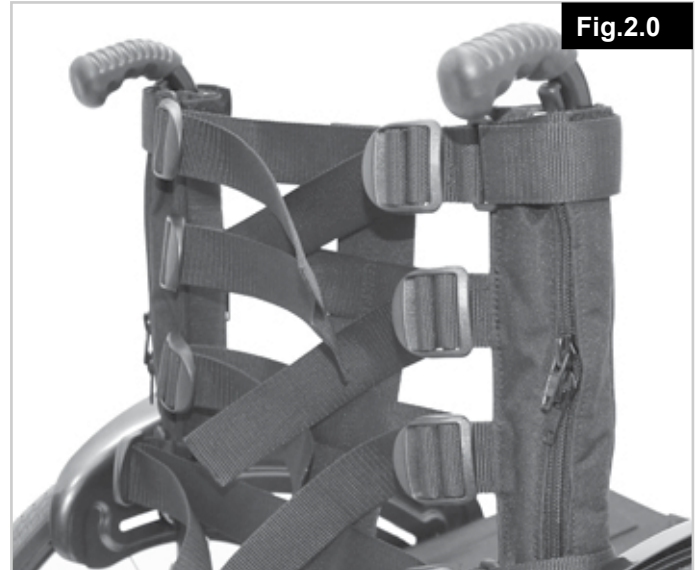**Fig.2.0****Fig.3.0****Fig.4.0**

ADJUSTMENT

To adjust the tension of the JAY Shape Back:

- Lift up the back cover flap to expose the horizontal straps.
- Adjust each strap individually to the desired tension.
- Fold down the back cover flap over the horizontal adjustment straps to complete your adjustment.

NOTE: You may adjust the JAY Shape Back while the user is seating on the chair.

CAUTION!

AVOID over-tightening of the strap assembly as this may cause damage to the integrity of the back canes.

Installing the lateral supports:

1. Determine the desired location of the lateral supports on the back tubes, (Fig.5.0).
2. Attach and align receiver on the back tubes, (Fig.5.0).
3. Repeat the process for the second lateral support if applicable.
4. Install the back cane covers and open the zippers to leave space for the lateral supports accordingly, (Fig.6.0).
5. To adjust the angle of the lateral supports, unscrew the bolts as described in the picture, (Fig.7.0).

Fig.5.0

Fig.6.0

Fig.7.0

MAINTENANCE AND CLEANING

Sunrise recommends that the Jay Shape Strap Assembly be checked for wear, such as loose, worn or broken components every 6 months. Loose straps should be retightened according to the installation instructions. Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose, worn or broken components.

EASY CLEANING

NOTE: When disassembling the back for cleaning, note the position of the Strap Assembly and cover for correct reassembly.

To clean the outer cover:

Wipe down the outer cover using a soft cloth and warm soap and water.

The outer cover can be machine washed in warm water (40°) and light detergent as needed.

NOTE: Remove foam from inside of cover prior to machine washing.

The foam core may be washed by hand with light soap and water.

Strap Assembly

Wipe down strap assembly as needed using light soap and warm water.

CAUTION!

Do not machine wash the strap assembly. If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

Laundry Care Symbols

	Machine wash in warm water: (40° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

JAY SHAPE BACK WARRANTY

Every Jay Shape Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the cushion's removable cover.

The removable cover is guaranteed to be free from defects in materials and workmanship for a period of six (6) months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within six (6) months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge.

Claims and repairs should be processed through the nearest authorised supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded.

There is no implied warranty beyond what is contained herein.

Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

This does not affect your statutory rights.

RECYCLING / RE-USE:**WARNING!****Risk of Injury.**

- Always have an authorized Sunrise Medical dealer inspect the product for damage or wear prior to transferring it to another user.
- If any type of damage is found, **DO NOT USE THIS PRODUCT.**
- Failure to heed these warnings may lead to severe injury.

This Product is suitable for re-use*:

- Have an authorized Sunrise Medical dealer inspect the strap assembly, foam and cover for damage or wear.
- If any damage is found, do not re-use.
- Clean and disinfect the strap assembly, foam and cover in accordance with local government regulations.
- Refer to your local Infection Control Office for advice.
- After cleaning and disinfection, have an authorized Sunrise Medical dealer inspect the strap assembly, foam and cover to insure that the cleaning/ disinfection process did not cause any damage.

CONTRAINDICATIONS:**WARNING!**

- Do not use or prescribe this backrest for anyone exceeding the stated Maximum User weight.

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Sunrise Medical Pty. Ltd.
Sunrise Medical GmbH & Co. KG