1930E H320 H330/I

F	1.	Guide utilisateur
GB	2.	User's manual
D	3.	Gebrauchsanweisung
E	4.	Manual del usuario
I	5.	Manuale dell'utente
NL	6.	Gebruiksaanwijzing
P	7.	Guia do utilizador
S	8.	Bruksanvisning
DK	9.	Brugsanvisning
FIN	10.	Käyttäjän käsikirja
N	11.	Bruksanvisning

1. FRANCAIS

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2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

1930E BATH ACCESS BAR

- Position the access bar over the sides of the bath with the sliding feet inside. Tighten the plastic screws.
- Check that the bar is installed correctly and that the black non-slip feet are in good condition before use.

H320 PLASTIC SUPPORT BAR

Place the support bar at the required location and mark the position of the fixing holes. Drill and fix with the appropriate plugs.

H330/I FOLD-AWAY SUPPORT BAR

Position the support bar beside the toilet. Fix to the wall with four 5x50mm screws and plugs.

MAINTENANCE

Clean regularly with a normal household cleaner. For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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